Adapter Chart (US)

Adapter Selection and Device Settings

Company & Device	Model	On/Off	Adapter Number
Blood Glucose Monitors			
Abbott Diabetes Care	FreeStyle	On	7
	FreeStyle Flash®	On	7
	FreeStyle Freedom®	On	7
	FreeStyle Lite™	On	7
	Precision Xtra®	On	10
	Precision QID®	On ⁴	5
Bayer Diabetes Care	Ascensia® Breeze®	On	1
	Ascensia® Breeze®2	On	1
	Ascensia® Contour(2 Button)	On	1
	Ascensia® Contour(3 Button)	On	1
	Ascensia® Elite® XL	On ⁵	1
	Glucometer® Elite® XL	On ⁵	1
HOMEdiagnostics™	Prestige IQ®	On	8
	TRUTrack® Smart System	Off	8
	TRUEread™	Off	8
LifeScan, Inc.	OneTouch® II	On	6
	OneTouch® Basic®	On	6
	OneTouch® FastTake®	Off ²	6
	OneTouch® InDuo®	Off	6
	OneTouch® Profile®	On	6
	OneTouch® SureStep®	On	6
	OneTouch® Ultra®	Off	6
	OneTouch® Ultra®2	Off	6
	OneTouch® UltraSmart®	On	6
	OneTouch® UltraMini™	Off	6
	OneTouch® Select™	Off	6
Roche Diagnostics	Accu-Chek® Active	On ³	11
	Accu-Chek® Aviva	On ³	11
	Accu-Chek® Compact	On ³	11
	Accu-Chek® Compact Plus	On ³	11
	Accu-Chek® Advantage	Off	3
	Accu-Chek® Complete	Off	4
Blood Pressure Monitors			
A&D Medical	LifeSource® UA-767PC - Arm	On	13
Omron®	HEM-637 - Wrist	On ¹	14
	HEM-670IT - Wrist	On ¹	14
	HEM-705CP - Arm	On	14
Weight Scale			
A&D Medical	LifeSource® UC-321PL	On ⁶	15
Pulse Ox❖			
Nonin®	Wrist Ox® 3100	On	18
PT/INR❖			
Hemosense®	INRatio™	On	20

If the device must be set to "On", begin the upload as soon as possible after turning it on or it may turn off automatically and prevent the upload from completing.

1. Press the Memory Button ("M") to turn on the device, not the "Start/Stop" button.
2. This monitor requires a One Touch FastTake adapter, which must be obtained from LifeScan in addition to a #6 adapter.
3. Set the device to the "data-transfer mode" to begin upload. See the device user guide for more information.
4. This device works with MetrikLink only in Computer Mode. See the *Uploading Data Using MetrikLink, PC Users* section in the MetrikLink user guide.
5. When a serial connection is used, make sure the connection is to Serial Port #1. If connection is good, three dashes (---) should appear on the monitor before uploading.
6. Set the device to Profile A to store data for upload.

❖ Caution: When connected to a prescription monitoring device, Federal law restricts the MetrikLink® device to sale by or on the order of a physician.

Note: We continually add MetrikLink support for new devices. For the current list of supported devices, go to <http://www.imetrikus.com> and download the PDF or call iMetrikus Customer Support at +1 888.253.5001.

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