

Adapter Chart (UK)

Adapter Selection and Device Settings

Company & Device Type	Model	On/Off	Adapter Number
Blood Glucose Monitors			
Abbott Diabetes Care	FreeStyle	On	7
	FreeStyle Mini™	On	7
	FreeStyle Freedom®	On	7
	Optium Xceed	On	10
Bayer Diabetes Care	Ascensia® Breeze®	On	1
	Ascensia® Breeze® 2	On	1
	Ascensia® Contour (2 Button)	On	1
	Bayer® Contour (3 Button)	On	1
HOME diagnostics, Inc.®	Prestige QX®	On	8
	TRUEtrack® Smart System	Off	8
LifeScan®	OneTouch® Ultra®	On	6
	OneTouch® Ultra® 2	Off	6
	OneTouch® UltraSmart®	On	6
	OneTouch® GlucoTouch®	On	6
Roche®	Accu-Chek® Active	On ²	11
	Accu-Chek® Aviva	On ²	11
	Accu-Chek® Compact	On ²	11
	Accu-Chek® Compact Plus	On ²	11
	Accu-Chek® Advantage	Off	3
	Accu-Chek® Complete	Off	4
Blood Pressure Monitors			
A&D Medical	LifeSource® UA-767PC - Arm	On	13
Omron®	R7 HEM-637 - Wrist	On ¹	14
Weight Scale			
A&D Medical	LifeSource® UC-321P	On ³	15
<p>If the device must be set to "On", begin the upload as soon as possible after turning it on or it may turn off automatically and prevent the upload from completing.</p> <ol style="list-style-type: none"> 1. Press the Memory Button ("M") to turn on the device, not the "Start/Stop" button. 2. Set the device to the "data-transfer mode" to begin upload. See the device user guide for more information. 3. Set the device to Profile A to store data for upload. 			

Note: We continually add MetrikLink support for new devices. For the current list of supported devices, go to <http://www.imetrikus.com> and download the PDF file or call iMetrikus Customer Support at 1-888-253-5001 for assistance.