MetrikLink[®] Adapter Chart (US)

Adapter Selection and Device Settings

Company & Device	Model	On/Off	Adapter Number
Blood Glucose Meters			
Abbott Diabetes Care	FreeStyle FreeStyle Flash [®] FreeStyle Freedom [®] FreeStyle Lite [™]	On	7
	Precision Xtra [®] Precision QID ^{® 1}	On On	10 5
Bayer Diabetes Care	Ascensia [®] Breeze [®] Ascensia [®] Breeze ^{® 2} Ascensia [®] Contour (2 button) Ascensia [®] Contour (3 button) Ascensia [®] Elite [®] XL ² Glucometer [®] Elite [®] XL ²	On	1
HOMEdiagnostics [™]	Prestige IO [®]	On	8
	TRUEtrack [®] Smart System TRUEread [™]	Off	8
LifeScan, Inc.	OneTouch [®] II OneTouch [®] Basic [®] OneTouch [®] Profile [®] OneTouch [®] SureStep [®] OneTouch [®] UltraSmart [®]	On	6
	OneTouch [®] Ultra [®] OneTouch [®] Ultra ^{® 2} OneTouch [®] UltraMini [™] OneTouch [®] InDuo [®] OneTouch [®] Select [™] OneTouch [®] FastTake ^{® 3}	Off	6
Roche Diagnostics	Accu-Chek [®] Active ⁴ Accu-Chek [®] Aviva ⁴ Accu-Chek [®] Compact ⁴ Accu-Chek [®] Compact Plus ⁴	On	11
	Accu-Chek [®] Advantage	Off	3
	Accu-Chek [®] Complete	Off	4
Blood Pressure Monitors			
A&D Medical	LifeSource [®] UA-767PC - Arm	On	13
Omron [®]	HEM-637 – Wrist ⁵ HEM-670IT – Wrist ⁵ HEM-705CP – Arm	On	14
Weight Scale			
A&D Medical	LifeSource [®] UC-321PL ⁶	On	15
Pulse Ox[◆]			
Nonin [®]	Wrist Ox [®] 3100	On	18
PT/INR[◆]			
Hemosense [®]	INRatio [™]	On	20
<p>If the device must be set to "On", begin the upload as soon as possible after turning it on or it may turn off automatically and prevent the upload from completing.</p> <ol style="list-style-type: none"> 1. This device works with MetrikLink only in Computer Mode. See the <i>Uploading Data Using MetrikLink, PC Users</i> section in the MetrikLink User Guide. 2. When a serial connection is used, make sure the connection is to Serial Port #1. If connection is good, three dashes (---) should appear on the monitor before uploading. 3. This monitor requires a One Touch FastTake adapter, which must be obtained from LifeScan in addition to a #6 adapter. 4. Set the device to the "data-transfer mode" to begin upload. See the device user guide for more information. 5. Press the Memory Button ("M") to turn on the device, not the "Start/Stop" button. 6. Set the device to Profile A to store data for upload. <p style="text-align: center;">◆◆ Caution: When connected to a prescription monitoring device, Federal Law restricts this device to sale by or on the order of a physician.</p>			

Note: We continually add MetrikLink support for new devices. For the current list of supported devices, go to <http://www.imetrikus.com> and download the PDF or call iMetrikus Customer Support at +1 888.253.5001.

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